



Total occlusion eye patch instructions

1. Position the silicone eye patch gently on the eye making sure to cover it completely. (The air holes should face outward.)
2. Maintain the eye patch in place with a finger and gently position the glasses to determine where the suction cup should be attached.
3. Remove the glasses and attach the suction cup at the spot place noted in the previous step. Press on both sides of the suction cup. Then make sure the eye patch is secured by lightly pulling it. For better adhesion, wet the suction cup.
4. Put the glasses on and make sure the eye patch completely covers the eye. Repeat steps 1, 2 and 3 if there is any discomfort at all.
5. When the treatment period is finished, take off the glasses and then gently remove the eye patch from the glass by wedging your fingernail under the suction cup.
Do not pull on the eye patch.
6. After use, gently wash the eye patch with soapy water, rinse and store it in the case provided for this purpose.

Visit www.drpatch.ca for more information.

